

Installation Instructions

Silverback Hardware



Blake
Medical Distribution

1-866-662-3459
www.blakemedical.ca




When you receive your Silverback from us the **Bracket** and **Pins** will already be installed on the Silverback Shell. The **Mounting Hardware** will come separately.

Installation Tools Required: 5 mm Allen Key & 2.5 mm Allen Key

To Install the **Mounting Hardware** to your wheelchair, follow the steps below.

1. Unscrew the **Clamp Screws** and remove the **Clamp** from the **Base** of the mounting hardware.
2. Take the **Rubber Grip** and wrap it around the back canes of your chair.
3. Place the **Base** on the **Rubber Grip**, make sure its on the inside of the back canes with the pin lock at the top facing forward.
4. Place the **Clamp** on the **Rubber Grip** and line up the screw holes with the ones on the **Base** and screw the **Clamp Screws** in tight.
5. To prevent rotation tighten the **Set Screw** to ensure the hardware will not rotate while in use.

Mounting Hardware

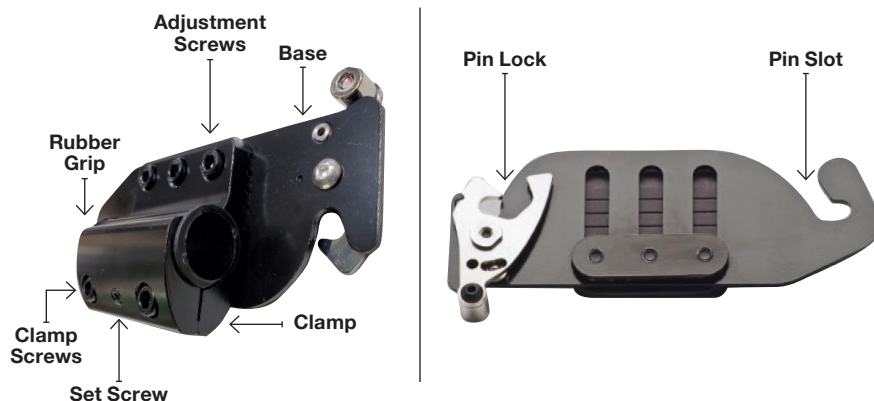

If you are receiving replacement hardware you will have to install the **Pins** and **Brackets**, follow the directions below.

1. Place the **Bracket** on the back of the shell with the open end facing away from the shell, place the **Stud Plate** through the front of the shell and the center holes on the **Bracket** so that the nut tightens on the back of the shell. Tighten the nuts.
2. Place the top **Pin** in the hole closest to you at the top of the **Bracket** and the bottom **Pin** in the back hole closest to the shell at the bottom of the **Bracket**.

To Install the **Backrest** onto the **Mounting Hardware**.

1. Place the Bottom **Pins** in the **Pin Slots**.
2. Once they are both in click the top **Pins** into the **Pin Locks** at the top of the **Mounting Hardware**. Back 2 picture shows proper installation.
3. When the back is securely in place you can then adjust its tilt position by loosening (do not completely take them out) the three **Adjustment Screws**. Adjust the back to your desired position and screw the **Adjustment Screws** back in tightly.
4. Your back is now ready for years of enjoyment!



Back 1



Back 2



Hardware Replacement Parts



Stud Plate & Bracket

Product Code: SBBS

Includes: A pair of brackets, two stud plates, 8 nuts and a set of 0.5" pins



Pins

Product Code:

0.5"PIN, 1"PIN, 1.5"PIN
Includes: A set of 4 pins of size specified



Mounting Hardware

Product Code: SBMH

Includes: A pair of Mounting Hardware for Silverback



Hardware Full Set

Product Code: SBHARDWARE

Includes: A pair of mounting hardware, a pair of brackets, two stud plates, 8 nuts and a set of 0.5" Pins