SOPHIA 2 / SOPHIA 4

Instructions and Warranty
Instructions et Garantie

*For questions or comments, please contact
our customer service.*

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orthex.ca / orthex.us

Table of content

4	CONGRATULATIONS!
6	PRODUCT
8	SECURITY
10	INSTALLATION
12	SOPHIA 2
13	SOPHIA 4
14	MAINTENANCE
15	TROUBLESHOOTING
16	WARRANTY

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CONGRATULATIONS!

Thank you for choosing the SOPHIA adjustable base.

This manual provides important information about the safe and proper use of your adjustable base. Please read this document thoroughly before using the adjustable base to optimize the base's performance and to ensure your safety. It combines two essential components: the user instructions and warranty information. The user instructions section provides detailed guidance on assembling, operating, and maintaining the adjustable base. The warranty section outlines the terms and conditions of our product warranty, including what is covered and any applicable limitations.

We understand the critical role adjustable bases play and we have designed our product with the utmost care and attention to quality and safety. By following the instructions outlined in this document, you can maximize the benefits of your adjustable base.

Please note this document is specific to the model and version of our adjustable base, as indicated on the product label. If you have any questions or require further assistance, please refer to the customer support section at the end of this document. We recommend that you keep this document for future reference. Once again, we appreciate your trust in our product(s) and are confident that the SOPHIA adjustable base will meet your needs and expectations.

Orthex continuously strives for excellence through innovation and collaboration, for this reason, it is our priority to provide help and support whenever you need. Please do not hesitate to contact us if you have any questions or comments.

Now, let's proceed with the comprehensive warranty and user instructions to help you make the most of your adjustable base.

OVERVIEW

	SOPHIA 2 Two positions adjustable base	SOPHIA 4 Multi positions adjustable base
Parts included	Adjustable base Wireless remote Retainers (2) Power supply Power cord Lithium battery pack	Adjustable base Wireless remote Retainers (2) Power supply Power cord Lithium battery pack
Functions	Head incline Feet incline Lumbar support Vibro-Massage Underbed lighting	Head incline Feet incline Lumbar support Vibro-Massage Height adjustment Trendelenburg Reverse Trendelenburg Underbed lighting Memory
Upholstery	Liquid resistant grey mix	Liquid resistant grey mix
Support structure	Legs	Casters
Dimensions, weight, capacity	Twin XL (38" x 80" x 12"), 140 lbs, 800 lbs Queen (60" x 80" x 12"), 200 lbs, 900 lbs	Twin XL (38" x 80" x 12"), 220 lbs, 800 lbs Double (54" x 74" x 12"), 290 lbs, 900 lbs Queen (60" x 80" x 12"), 320 lbs, 900 lbs

FEATURES DETAILS

Head incline : Can be inclined to an upright position of up to 68 degrees, providing optimal comfort and support while helping with digestion, breathing and heart conditions.

Feet incline : Can be raised to an elevated position of up to 9 inches, aiding with back pain, knee pain and blood circulation.

Lumbar support : Can be adjusted up to 4 inches to provide targeted and personalized support to the lower back area.

Vibro-massage : Offers gentle vibrations around the head and legs to alleviate stress and muscle tension with three intensity varieties.

Height adjustment : Can be adjusted to the user's preferences, ensuring optimal mobility and the convenience needed for specific medical requirements, within a range of 12 to 30 inches (excluding the mattress).

Trendelenburg and Reverse Trendelenburg : Allows you to tilt the bed's sleeping surface either head-up or feet-up, with a maximum tilt of 16 degrees to improve blood circulation, digestion and breathing while ensuring optimal body alignment.

Security

Electricity

- The connection socket must comply with the specifications and be properly grounded.
Input: AC 100V / 240V - 50 / 60Hz - 1.6A
Output: DC 24V - 2A
- Plugging the base into a surge protector is strongly recommended.
- No adapters should be used with this product.
- Before unplugging the base, make sure it is in a flat position with all motors off.
- Inspect the power cord regularly for any signs of damage or wear and do not use the base if the power cord is damaged.
- Avoid exposing the base to liquids or moisture and keep electrical components dry and away from heated sources.
- Avoid using the base and contact a qualified technician if the electrical cord or plug is damaged.

Improper connection of equipment-grounding conductor could cause an electrical shock. Please consult a qualified electrician or technician if you are unsure whether the product is properly grounded. Do not modify the plug provided with the product. If it does not fit properly into your outlet, have a proper outlet installed by an electrician. The use of an improperly grounded electrical box may compromise the operation of the safety back-up and may cause electrical shocks, fire from an electrical source or product malfunction.

Mechanism

- Only use the base for its intended use as described in this manual.
- Make sure there are no obstructions or that no one can be injured before activating the mechanism.
- Do not force the base down or tilt it on the side.
- Be sure to use a suitable mattress with this base.
- You may sit on the ends (corners) of the bed only when it is in the flat position.
- Do not modify or add parts that do not come from Orthex.

Environment

- This base should be placed directly on a solid floor with sufficient space.
- Do not push or pull the base by grabbing the side rails or any movable components.
- When moving the base, ensure that the casters are unlocked.
- Respect the specified maximum weight capacity to prevent damages to the base frame or injury to the patient or caregiver.
- If using additional accessories or equipment, ensure that the combined weight does not exceed capacity.
- Validate with manufacturer before using or adding accessories or attachments.
- Always use this product indoors and limit direct sunlight.
- Do not leave children or animals with the base unattended.

Additional notices

- Consult a health professional if you are using a magnetic sensitive device (Pacemaker, ICD).
- Casters may still move even locked.
- The massage units will emit a vibrating sound when activated.
- The material may have a smell when product is new, but will disappear rapidly.

INSTALLATION AND INSTRUCTIONS

1. Unpack all the components

- Choose a suitable location for the base assembly, ensuring there is ample space to maneuver and access all sides of the base.
- Carefully unpack all the components from the packaging.
- Validate that you have received all the parts listed in the parts list.

2. Position the adjustable base

- Validate that the desired location has sufficient clearance from walls or other furniture and valid electrical outlets.
- Maintain the adjustable base upside-down.

3. Connect electrical components

- Ensure all security guidelines are met before proceeding.
- Locate the electrical control box and battery backup on the adjustable base.
- Connect the power cord / power supply / lithium power pack to the control box and plug the other end into a grounded electrical outlet.

4. Pairing the remote

- Install two AAA batteries into the remote's battery compartment. Push in firmly to secure.
- Press Connect Button on the control box until the LED flashes, then press PAIR until the backlight LED flashes, release and press BED UP. Do not release until backlight LED is fully activated.
- Pairing is completed when the pairing button on the control box stops flashing and the buzzer on the control box has a DI sound.
- Ensure all electrical functions, including height adjustment, backrest, leg rest, and any additional features, are functioning.

5. Install the side rails and mattress retainers

- The base must always rest on all four wheels/legs.
- Turn base over securely (two people are required to avoid injuries or breakage).
- Align the side rails with the designated attachment points on the base frame.
- Compress both ends into the attachment points and ensure they are properly locked in place.

6. Verify stability and safety

- Ensure that all connections, retainers, and components are securely in place.
- Confirm that the base is stable and does not wobble or shake when pressure is applied.
- Double-check that all side rails are raised, stable and locked in position.

7. Dispose of packaging


- All packaging material must be discarded and kept out of reach of children.
- Dispose of all packaging materials according to local waste management guidelines.

8. Memory function (SOPHIA 4)


- Adjust your bed to your preferred position.
- Make any necessary fine-tuning adjustments to ensure maximum comfort.
- Once you've found your desired positions, press and hold the «PAIR» button on your remote control for three seconds and then press the «M» button.
- The memory position is now saved and accessible by simply pressing the «M» button.
- Periodically check and readjust your saved positions to accommodate any changes in your comfort preferences and/or needs.

SOPHIA 2

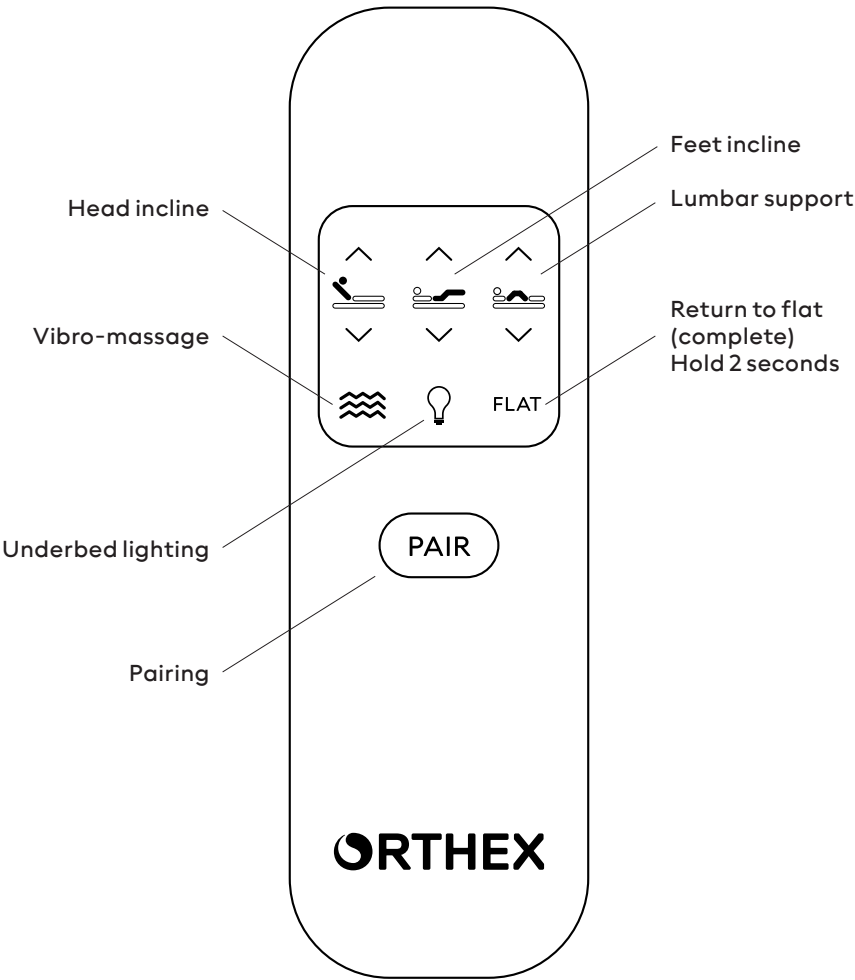Two positions ajustable base

SOPHIA 4
Multi-positions ajustable base

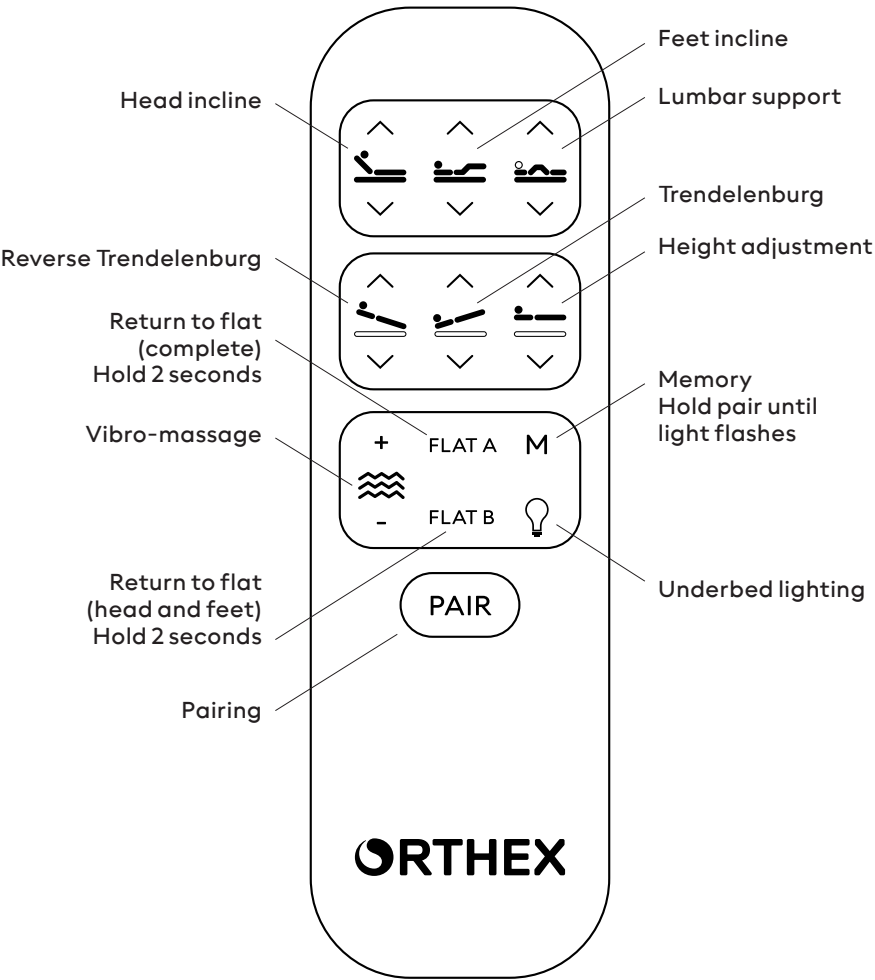MAINTENANCE

Fabric care

- To prolong the life of your base's fabric, protect it from direct sunlight whenever possible.
- For spot cleaning, blot dirty area with a light damp sponge or a cloth.
- Do not use harsh or abrasive cleaners that may damage the base's surfaces or finishes.
- Keep at a minimum of 30cm (12 inches) away from direct heat sources.

Mattress care

- Follow the manufacturer's instructions for cleaning and maintaining the mattress.
- Use mattress protectors or covers to prevent spills, stains, or damage to the mattress.

Electrical safety

- Regularly check the power cord and control panel for any signs of damage. Do not use the base if the power cord is frayed or damaged.
- If the control panel or electrical components are not functioning correctly, contact customer support for guidance.

Inspection

- Periodically inspect the base frame, side rails, retainers, and electrical components for any signs of damage, wear, or malfunction.
- Ensure that all bolts, screws, and connections are tight and secure.
- If you notice any issues or abnormalities, discontinue the use of the base and contact customer support for assistance.

TROUBLESHOOTING

Base does not respond to controls

- Ensure that the base is properly plugged into a functioning electrical outlet.
- Confirm the remote control is functional.
- Check the power cord for any damage or loose connections.
- Make sure the light on the control box flashes when activating the remote.
- Turn off the power and restart after 30 seconds.
- Press SW button for three seconds for a power reset.

Power outage

- The emergency power outage feature is seamlessly integrated into the bed's control system. When the main power source is disrupted, the lithium battery pack automatically engages, allowing you to use the bed's adjustable functions, for up to 30 cycles on a single full charge.

Base movements are shaky or uneven

- Check for any obstructions, debris or excessive loads hindering the base's movement.
- Inspect the casters for proper alignment and smooth rotation.
- Ensure that the base is on a flat, level surface (floor).

Base emits unusual noises

- Inspect the base for loose components or retainers and tighten as necessary.
- Check for any foreign objects or debris causing the noise and remove them.
- Validate if the noise comes from a motor and contact customer support for assistance if it's the case.

WARRANTY

Warranty period

- Three (3) years, from the date of purchase, on the electrical/electronic components.
- Lifetime, from the date of purchase, on the structure.

If your adjustable base or parts are found to be defective during the period prescribed by the warranty, Orthex will replace it/them with the same model or its equivalent. Replacement may also be provided in the form of credit for other Orthex products. In order to claim your warranty, you must provide proof of purchase (at an authorized Orthex retailer). Orthex reserves the right to repair or replace certain parts rather than the entire product.

This warranty does not cover:

- Damages or breakage caused by the owner.
- Damages or breakage caused by improper installation.
- Damages or breakage caused by improper use.
- Damages or breakage caused by improper handling.
- Damages or breakage caused by unauthorized parts.
- Damages or breakage caused by a mattress that does not meet the required specifications.
- Damages or breakage caused by excessive weight.
- Service or repairs made by a technician not authorized by Orthex.
- Noises from normal operation of the base.

Repairs or replacement of parts made under the terms of this warranty apply based on the original date of purchase and do not extend the duration of the warranty. If parts are to be replaced outside of the terms of this warranty, parts and labor will be charged.

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