

5300 ErgoGlide®



The Handicare ErgoGlide® one-way glide is used to help maintain good sitting position by allowing for easy repositioning of patients in chairs.

The friction reducing surface minimizes forces required to achieve a proper seated position for patients. The locking material reduces the risk of the patient slipping forward.

ErgoSlide® Dimensions
17.75" x 17.75" (45 x 45 cm)

Handicare

485 Millway Avenue Unit 2, Concord ON L4K 3V4 Canada | 1.877.304.5438 | www.handicare.ca
10888 Metro Court, St. Louis, MO 63043 USA | 1.866.891.6502 | www.handicareusa.com

5300 ErgoGlide®

Use Instructions:

If the patient is placed in a wheelchair by a mechanical lift:

1. Place the ErgoGlide® with the label visible and the arrow pointing toward the back of the chair to ensure proper placement of Glide.
2. When the patient is placed all the way in the chair, the ErgoGlide® will keep the patient from sliding down in the chair.

If the patient is getting in the chair on their own and cannot get to the back of the chair:

1. With the ErgoGlide's label visible and the arrow pointing toward the back of the chair, place the ErgoGlide® a few inches from the back of the chair.
2. Once the patient is in sitting position, the patient can be assisted back in to the chair with minimal effort.

Regular assessment of the patient's skin is essential if using the ErgoGlide® for extended periods of time. Consultations with a Registered Occupational or Physical Therapist is advised if the ErgoSeat is being considered for use with patients who have or are at risk of developing a pressure ulcer.

Important Prior To Usage - this is only a guide, as alternative methods may be used for patient handling. We strongly recommend that your facility administrator or risk manager review and approve all protocols used for safe patient handling.

Prior to using any equipment, be sure to check for signs of wear to ensure proper functionality. When using the ErgoGlide® for the first time, practice with a fellow caregiver until you have a level of comfort with the product.

Laundry Instructions: The ErgoGlide® can be machine-washed at 80°C/170°F. Hang to dry or tumble dry on low heat at 40°C/104°F.

Handicare

485 Millway Avenue Unit 2, Concord ON L4K 3V4 Canada | 1.877.304.5438 | www.handicare.ca
10888 Metro Court, St. Louis, MO 63043 USA | 1.866.891.6502 | www.handicareusa.com