

5200 ErgoSeat®



The Handicare ErgoSeat® is a revolutionary 2-piece seating device for repositioning and standing assistance.

There are no restrictions to movement, as the ErgoSeat® offers omni directional options, meaning the patient can move laterally, in a circular motion or prepare to go from sit to stand.

The ErgoSeat® has adjustable straps and a durable buckle to easily secure to most seats or chairs.

ErgoSeat® Dimensions
22.5" x 21.5" (57 x 54.5 cm)

Handicare

485 Millway Avenue Unit 2, Concord ON L4K 3V4 Canada | 1.877.304.5438 | www.handicare.ca
10888 Metro Court, St. Louis, MO 63043 USA | 1.866.891.6502 | www.handicareusa.com

5200 ErgoSeat®

Use Instructions:

Place the ErgoSeat® with the shiny sides together and the softer sides to the outside. Secure the ErgoSeat® to the chair or seat using the strap and buckle.

Regular assessment of the patient's skin is essential if using the ErgoSeat® for extended periods of time. Consultations with a Registered Occupational or Physical Therapist is advised if the ErgoSeat is being considered for use with patients who have or are at risk of developing a pressure ulcer.

Important Prior To Usage - this is only a guide, as alternative methods may be used for patient handling. We strongly recommend that your facility administrator or risk manager review and approve all protocols used for safe patient handling.

Prior to using any equipment, be sure to check for signs of wear to ensure proper functionality. When using the ErgoSeat® for the first time, practice with a fellow caregiver until you have a level of comfort with the product.

Laundry Instructions: The ErgoSeat® can be machine-washed at 80°C/170°F. Hang to dry or tumble dry on low heat at 40°C/104°F.

Handicare

485 Millway Avenue Unit 2, Concord ON L4K 3V4 Canada | 1.877.304.5438 | www.handicare.ca
10888 Metro Court, St. Louis, MO 63043 USA | 1.866.891.6502 | www.handicareusa.com