


HighStar Healthcare Inc. Sling Instruction Sheet

TRI-TURNER SLING

Conducting a thorough risk assessment is essential to ensuring the appropriate size, material and style of sling is used

HighStar's Tri-Turner Sling is designed for toileting with maximum support. Featuring back and head support as an option, its self-adjusting waist strap allows for precise and comfortable positioning during a transfer. Offered in highly washable woven fabric, the Tri-Turner Sling can be used with any compatible floor lift or ceiling lift that accepts loop-style slings.

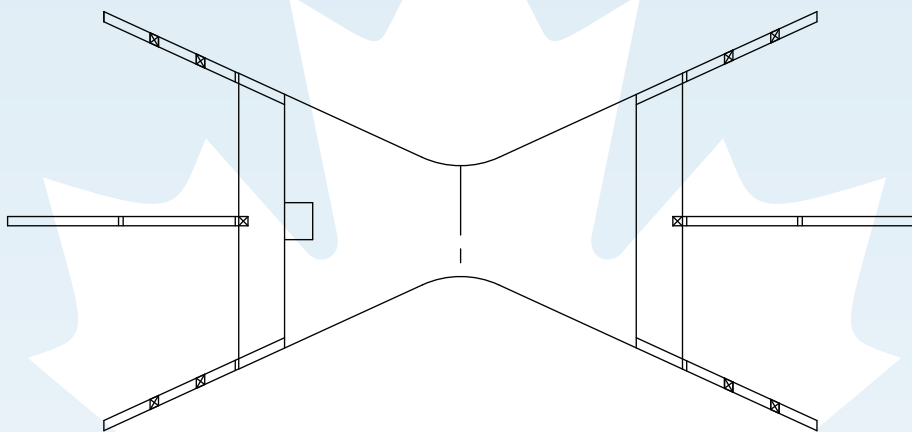

Washing Guidelines

- Wash inside a laundry bag for a longer-lasting sling
- Temperatures not to exceed 80°C/ 176°F
- Do not use bleach
- Air dry, or cool tumble dry. No heat.
- Do not dry clean
- Do not iron

Always check the product label for washing instructions.



General Guidelines

- Read and understand the instructions before use
- Always visually inspect the sling before use, checking for tears, rips, or frays.
- Do not exceed the sling's weight capacity
- Check the label is legible
- A properly cared for sling should approximately last 2-3 years


Remove from use if the sling appears to be deteriorating or if the label is no longer legible



INSTRUCTIONS FOR SLING APPLICATION

Always check the appropriate sling style, material and size has been selected before applying the sling

- Always ensure the sling's label is on the outside and remove applicable clothing.

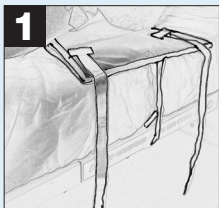

Push the Strap End Stitched with Velcro loop and hook of the Tri Turner Sling under the lower back of the patient by compressing the mattress.

Slide the strap to the other side of the bed frame.




Open the Velcro Loop and attach it to the bed frame.

Attach the Tri Turner Sling Straps to the Lift hooks. Start Lifting the Sling. Make sure the Lift hooks are parallel to the patient's back Side. Cross or straighten legs before turning.



Note: Support the Patient while turning. Support Patient's back with pillow when giving bath.

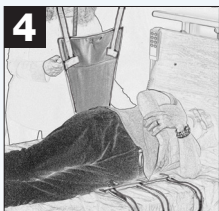

After completing turning of the patient, lower the sling. Remove the Vel-cro loop and hook from the bed frame and fasten it onto itself before you pull it out from underneath the patient.

- Before raising, each of the sling's loops must be securely fastened to the carry bar.

