



Instructions of Use

In-Situ Deluxe Leg

IN-SITU RANGE

Reference: **IS-DLS-IOU**

Product Codes: **M921511**


Revision: **06.09.2024**

Copyright and Design Rights: **Silvalea Ltd**





WARNING

Do NOT use clip sling attachments on a loop type spreader bar and Do NOT use loop sling attachments on a stud type spreader bar.

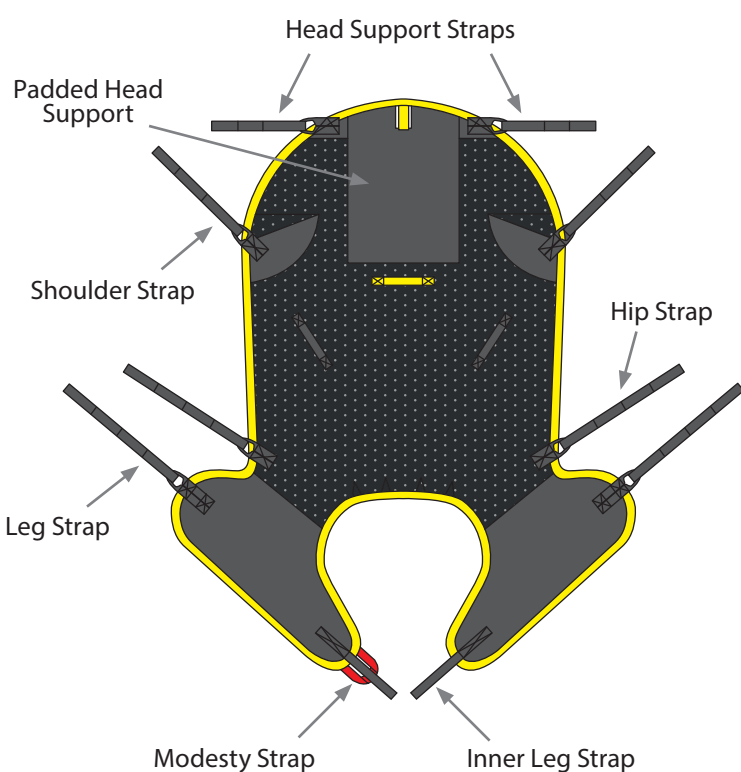

Loop Sling Attachments - Loop Type Spreader Bars:


Please note there are many variations of the loop type spreader bar but they should all be used exclusively by slings with: **LOOP SLING ATTACHMENTS**

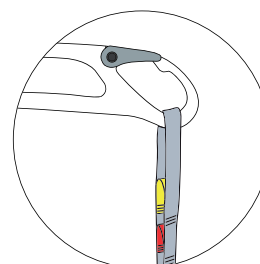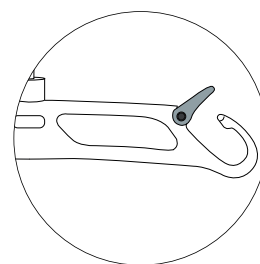


WARNING: Only use with suitable LOOP lift:




Sling Application:

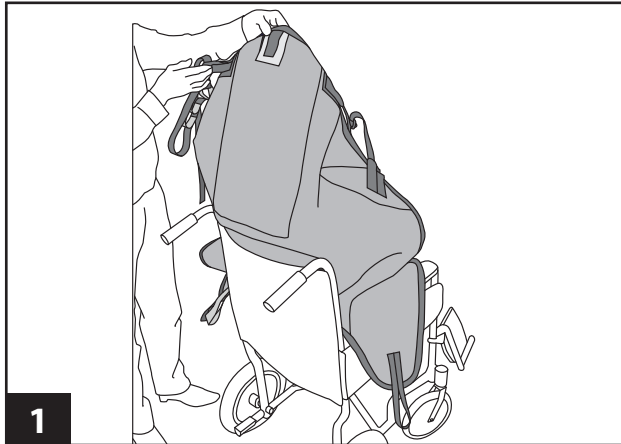


1 With sling label facing outwards glide sling down back of patient.

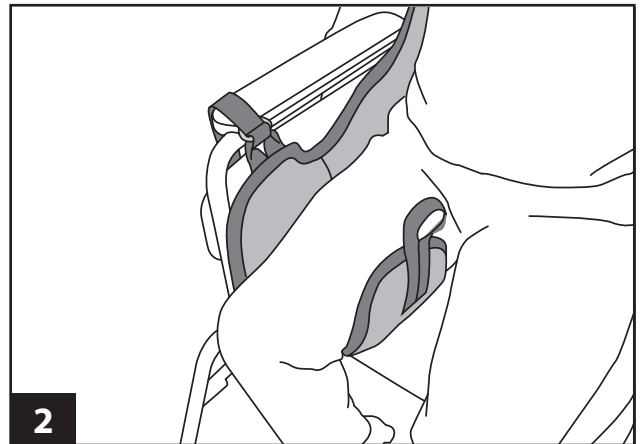


2 Place each leg section down the side and under each thigh. Check it is not twisted.

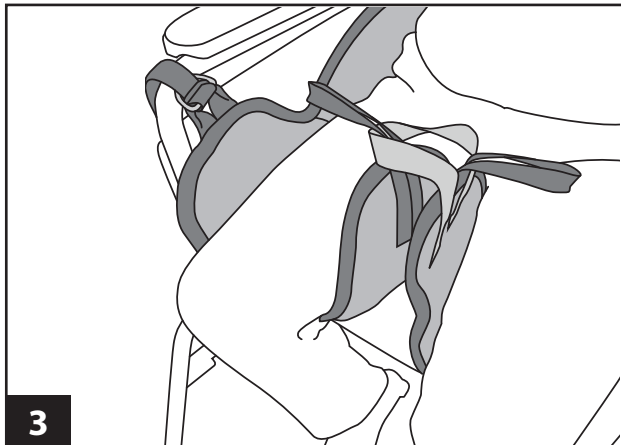

3 Thread the shorter inner leg strap through the colored modesty strap to prevent patient's legs abducting.

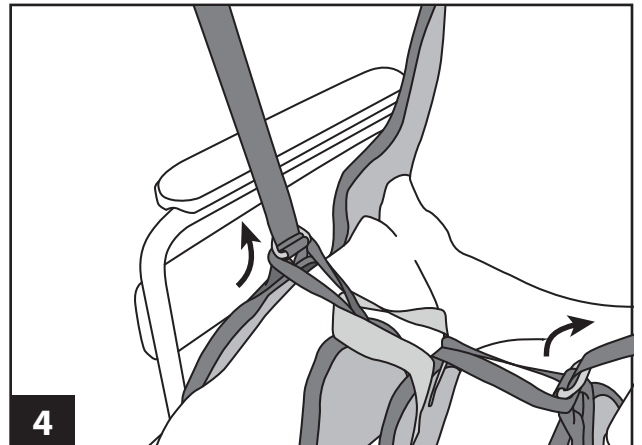


4 Thread the longer leg straps through the shorter inner leg straps as shown.

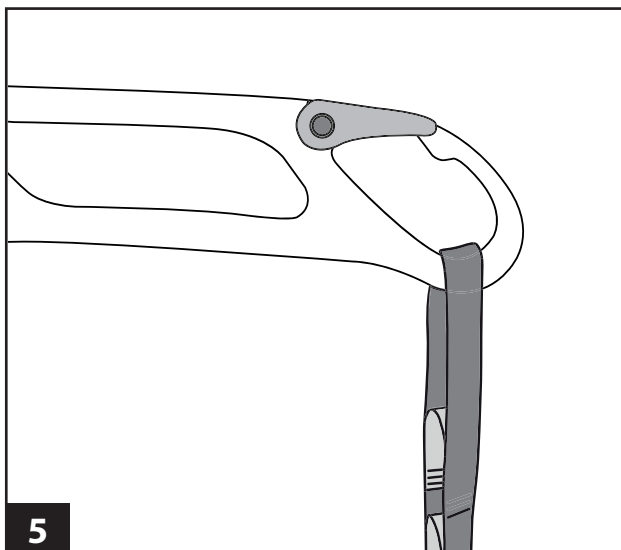

5

Once you are happy that the sling has been correctly fitted. Attach the two shoulder straps onto the carry bar of the lift and then the two leg straps.


Color coding on the destination loops are provided for ease of reference and position.

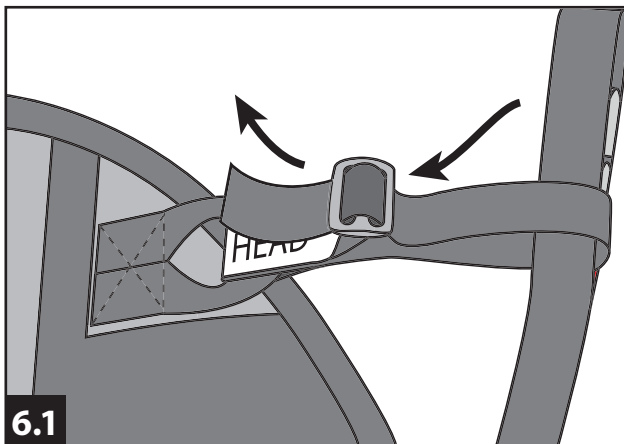

6.1

Either 6.1, feed the head support strap through the shoulder strap of the sling.


Then loop the head support strap back through the plastic slider correctly. Adjust the head support to suit patient's needs by adjusting the amount of strap passed through the slider.

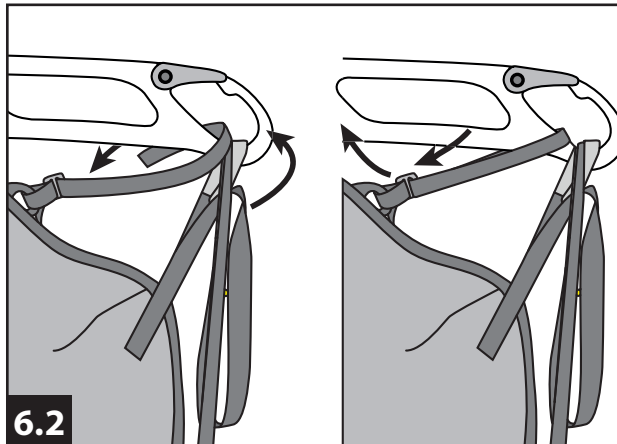

6.2

Or 6.2, attach head support strap directly to the lift carry bar.


Then loop the head support strap back through the plastic slider correctly. Adjust the head support to suit patient's needs by adjusting the amount of strap passed through the slider.

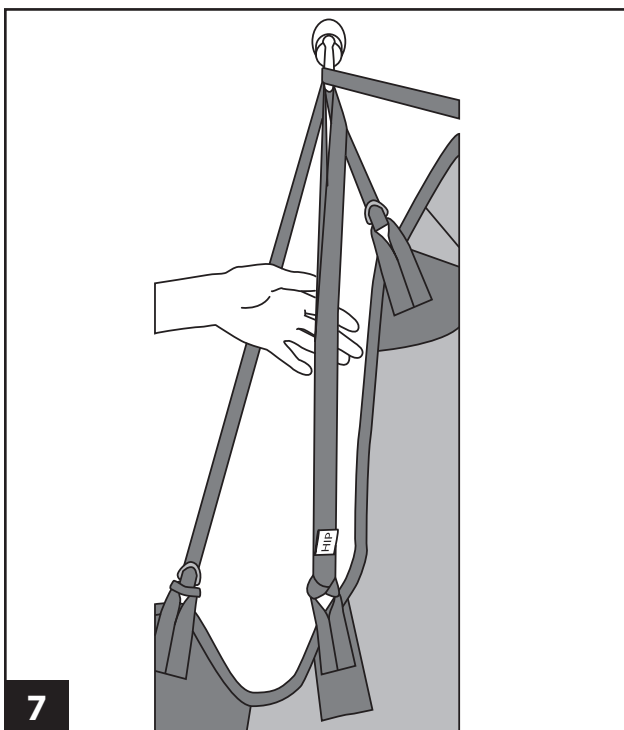

7

When attaching hip tapes, the ideal pick up should enable the pelvic area and the legs to pick-up simultaneously.

Adjustment can be made by using the color coded destination loops to enable the best pick-up.

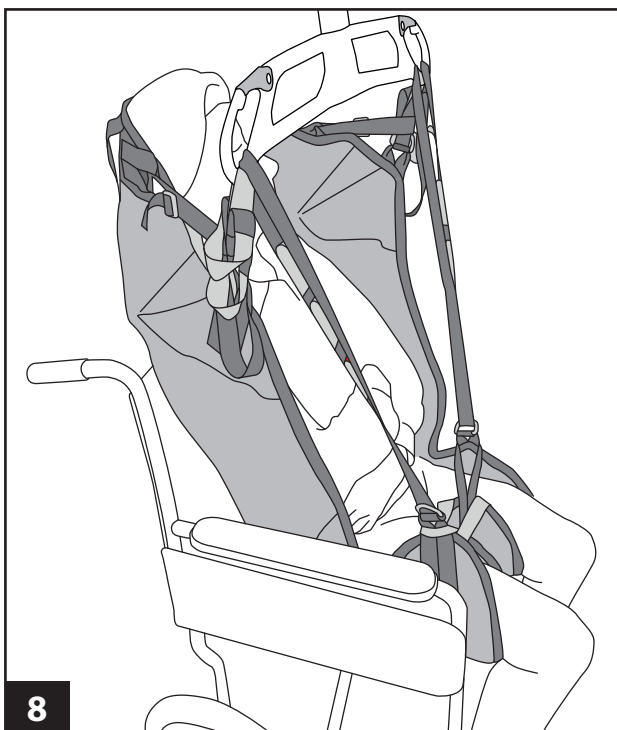

8

Ensure that all straps are not twisted and are attached securely before commencing the transfer.

NOTE: Once the sling takes the weight of the patient you may need to re-adjust the position of the sling for the patient's comfort. Lower the patient back onto the chair/bed to make any adjustments before continuing the transfer. Never attempt to lift your patient from a supine position unless a head support is fitted to the sling.



General Instructions: WARNING

Do not attempt to transfer the patient until all appropriate sling attachments are secured to the lift spreader bar and you are satisfied that the sling is the correct style, size and fit for the patient.

Correct sizing of a sling is important as a sling that is too large could cause the patient to slip through the aperture.

While every attempt is made to instruct a safe move you must conform to the current moving and handling procedures/regulations. If in any doubt, consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products shall be checked each time they are used for any signs of deterioration.

The service label is a visual guide to the product's check. Please conduct a thorough sling check in line with current safety requirements in your country. If in doubt consult your distributor for current sling checking protocols.

A risk assessment must be carried out to ensure that the correct size and style of sling has been met before use.

Pre-use sling check list:

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the lift
- safe working load of both the lift and sling/product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label. Please note: temperatures advised are MAXIMUM washing temperatures.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products shall be laundered using non- biological detergents.
- DO NOT IRON

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label. Please note: temperatures advised are MAXIMUM drying temperatures.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convactor heater or steam pipes.



SWL 272kg/600lb/42st

For sizing guides and images of this product please visit www.savariapatientcare.com and for further assistance, contact your Silvalea distributor



Silvalea Ltd a division of **Savaria**

Silverhills Buildings, Decoy Ind. Estate,
Newton Abbot, Devon, TQ12 5LZ. UK



Distributed in North America by Savaria (savaria.com) and the following Savaria divisions:



handicare

handicare.ca | handicareusa.com



spanamerica.com | Span Medical Products Canada



ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta