



Universal Sling product guide

SPAN

 **SILVALEA®**



Since 1993, Silvalea has been at the forefront of the patient sling market, designing and manufacturing a wide portfolio of solutions for stand-aid slings, patient transfer aids, in-situ slings, specialty, and complex patient transfer, along with an everyday range of replacement transfer slings. The comprehensive offerings of Silvalea are founded in over 25 years of innovation in fabric and fastening systems designed to address the specific needs of patient and caregiver alike.

Today, Silvalea is proudly part of Savaria Corporation with North American distribution of its products through Span. A wide range of Silvalea-designed slings will be manufactured at Span's U.S. facility offering fast lead times and locally made products. Whether the need is complex or simple, Silvalea has a solution.



The Silvalea universal sling series has been selected and manufactured specifically to be interchangeable with a variety of lifts. The products are intended as replacement slings that are easy to fit while providing superior support.



About Span

For over 30 years, Span-America has offered the industry's most comprehensive line of specialty solutions for pressure management and patient positioning. With an emphasis on clinical solutions, Span focuses on positive patient outcomes. The company proudly manufactures in Greenville, South Carolina (surfaces) and Beamsville, Ontario (beds). Acquired by Savaria Corporation in 2017, offerings were expanded to include Savaria patient lifts. Innovatively designed with better battery technology, a SureClip system for improved attachment safety, and overall simplicity of operation, these lifts and ceiling track systems provide industry-leading benefits for both institutional and home use. With the addition of Silvalea slings in 2019, Span now delivers a powerful portfolio of patient handling solutions.



About Savaria


Today a global leader in accessibility, Savaria designs and builds a vast range of wheelchair lifts, stairlifts, home elevators and specialty commercial elevators for accessibility. The company employs over 1,500 people around the world including 30 staff in research and development dedicated to creating new products and innovations for improving existing products. Publicly traded on the Toronto stock exchange (TSX:SIS), Savaria is a financially strong company dedicated to improving mobility for people in and out of the home.



Universal Sling with Head Support

WITH LOOPS

- Adjustable integral padded head support
- Full body supportive sling
- Divided padded leg sections
- Ideal for general transfers
- Small aperture for hygiene purposes
- Color coded tapes
- **Mesh** (100% polyester open weave mesh; high density, durable, used for bathing as quick drying.)

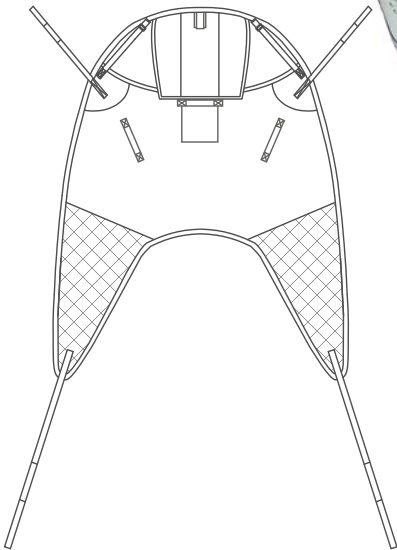

A




Shoulder circumference

B

Top of head to base of spine

Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921500-XS
	Small	34–37.75in (86–96cm)	26–29in (66–74cm)	M921500-S
	Medium	37.75–42.5in (96–108cm)	29–31.5in (74–80cm)	M921500-M
	Large	42.5–51in (108–130cm)	31.5–34.5in (80–88cm)	M921500-L
	XLarge	51–56.5in (130–144cm)	34.5–39.25in (88–100cm)	M921500-XL
	XXLarge	56.5–63in (144–160cm)	37.75–41in (96–104cm)	M921500-XXL

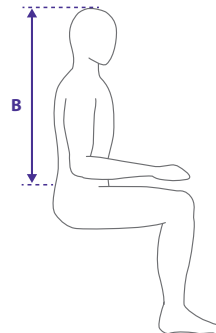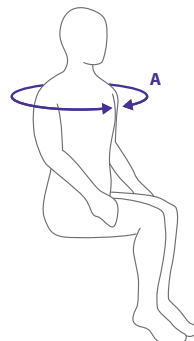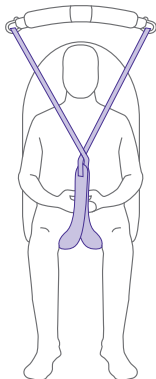

Guidelines


- SWL 272 kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry

Universal Sling without Head Support

WITH LOOPS



- Full body supportive sling
- Divided padded leg sections
- Ideal for general transfers
- Small aperture for hygiene purposes
- Colour coded tapes
- **Mesh** (100% polyester open weave mesh; high density, durable, used for bathing as quick drying.)

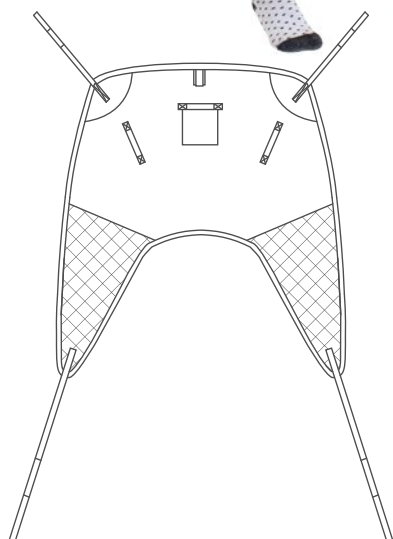

A




Shoulder circumference

B

C7 to base of spine

Product Codes

XSmall	30–34in (76–86cm)	17.5–19.75in (44–50cm)	M921501-XS
Small	34–38in (86–97cm)	19.75–22.5in (50–57cm)	M921501-S
Medium	38–42.5in (97–108cm)	22.5–25.25in (57–64cm)	M921501-M
Large	42.5–51.25in (108–130cm)	25.25–27.25in (64–69cm)	M921501-L
XLarge	51.25–56.75in (130–144cm)	27.25–29.25in (69–74cm)	M921501-XL
XXLarge	56.75–63in (144–160cm)	29.25–31.5in (74–80cm)	M921501-XXL

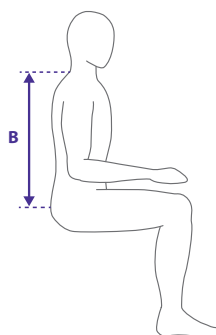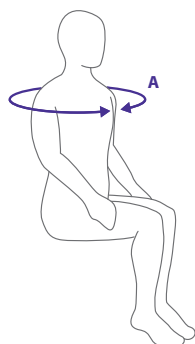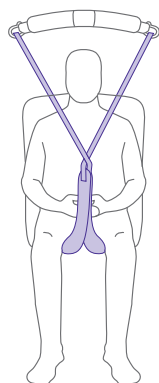

Guidelines

- SWL 272 kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



Hygienic Sling

DRESSING & TOILETING

The Hygienic sling with padded rolls has been designed with maximum comfort in mind.

- The combination of underarm padded rolls and full padding throughout offers greater user comfort.
- Used for dressing and toileting, this sling is suited to those with good upper body control.
- Features a waist support with buckle for additional support and safety.
- As with all hygienic slings, it is especially important that this sling is fitted correctly due to the aperture. A full patient risk assessment is required.
- **Poly** (100% polyester; easy application, glides and slides smoothly, dries quickly after washing.)

A



Waist circumference

B

Armpit to Waist

Product Codes

	XSmall	30.3–34.3in (77–87cm)	8.7in (22cm) or over	M921502-XS
	Small	34.3–42.1in (87–107cm)	9.8in (25cm) or over	M921502-S
	Medium	38.2–46.1in (97–117cm)	10.6in (27cm) or over	M921502-M
	Large	44.1–51.2in (112–132cm)	11.0in (28cm) or over	M921502-L
	XLarge	49.2–57.1in (125–145cm)	11.8in (30cm) or over	M921502-XL
	XXLarge	54.3–62.2in (138–158cm)	13.4in (34cm) or over	M921502-XXL

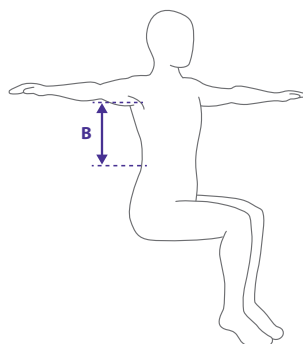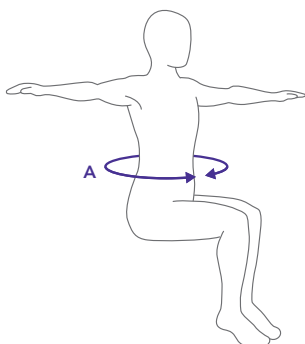

Guidelines

- SWL 272kg / 600lb
- Wash at 85°C
- Tumble dry low heat (max 50°C)

High Easy Sling


POLY

SUPERFINE PLUS™

WITH LOOPS

- This full body sling transfers in a slightly reclined position and is extremely comfortable and supportive, featuring additional padding on the leg sections.
- The sling is simple and easy to apply and is a go-to sling for many institutions due to its ease of application and minimum options. Includes attached hip tapes as standard for optimum client comfort.
- This sling is suitable for use with the leading standard spreader bar type lifts (loop fixing). Please see the compatibility statement at the back of this guide.
- Available in the following material types:
Superfine PLUS™ (100% polyester spacer fabric; superior comfort over standard sling fabrics; easy application, suitable for bathing.)
Mesh (100% polyester open weave mesh; high density, durable, used for bathing as quick drying.)

* Also available in Poly (M921514-XS; M921514-S; M921514-M; M921514-L; M921514-XL and M921514-XXL).



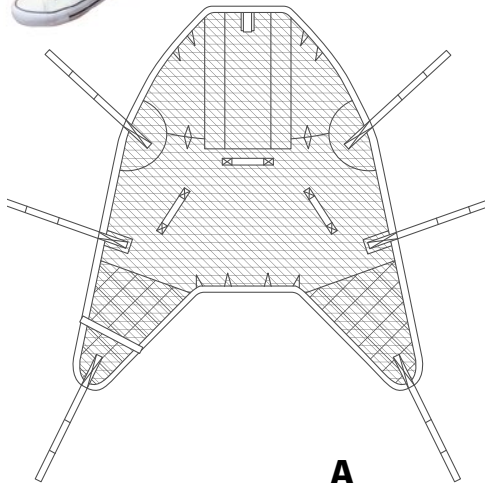


A
Shoulder circumference

B
Top of head to base of spine

Product Codes

				Superfine PLUS™	Mesh
	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921521-XS	M921513-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921521-S	M921513-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921521-M	M921513-M
	Large	42.5–51.25in (108–130cm)	31.5–34.5in (80–88cm)	M921521-L	M921513-L
	XLarge	51.25–56.75in (130–144cm)	34.5–39.5in (88–100cm)	M921521-XL	M921513-XL
	XXLarge	56.75–63in (144–160cm)	38–41in (96–104cm)	M921521-XXL	M921513-XXL

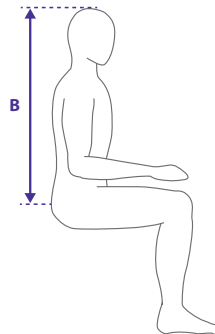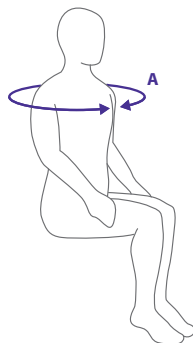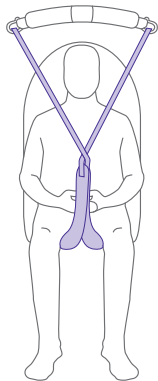

Guidelines


- SWL 272kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



High Easy Sling

WITH UNIVERSAL SAFETY CLIPS

- The full body sling transfers in a slightly reclined position and is extremely comfortable and supportive, featuring additional padding on the leg sections.
- The sling is simple and easy to apply and is a go-to sling for many institutions due to its ease of application and minimum options.
- The universal safety clips ensure an easy and safe application, with the ability to interchange among the leading standard stud spreader bar type lifts (clip fixing). Please see compatibility statement at the back of this guide.
- The sling includes tailoring at the shoulders and commode area to provide the optimum comfort fit.
- **Poly** (100% polyester; easy application, glides and slides smoothly, dries quickly after washing.)

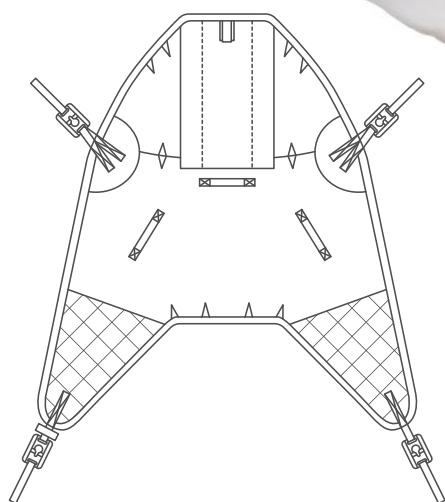

A




Shoulder circumference

B

Top of head to base of spine

Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921515-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921515-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921515-M
	Large	42.5–51.25in (108–130cm)	31.5–34.5in (80–88cm)	M921515-L
	XLarge	51.25–56.75in (130–144cm)	34.5–39.5in (88–100cm)	M921515-XL
	XXLarge	56.75–63in (144–160cm)	38–41in (96–104cm)	M921515-XXL

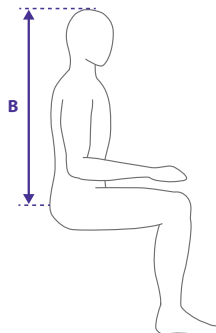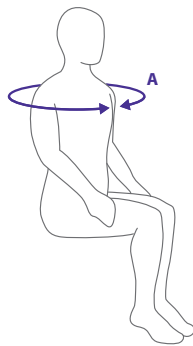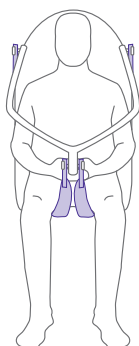

Guidelines


- SWL 272kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



Fastfit Deluxe Hammock Sling

WITH LOOPS

- The Fastfit Deluxe Hammock sling is a full-body supporting patient transfer sling with divided leg and integral head support. This sling is suitable for a wide range of user scenarios.
- Leg sections have been shaped to allow for additional support to the thigh and hip area.
- Small aperture for toileting. The Fastfit Deluxe comes with good back and leg support as well as hip tapes as standard.
- This sling is particularly suited for patients with arthritis and/or hip issues.
- Suitable for geriatric user transfers as well as general user transfers.
- **Superfine PLUS™** (100% polyester spacer fabric; superior comfort over standard sling fabrics; easy application, suitable for bathing.)

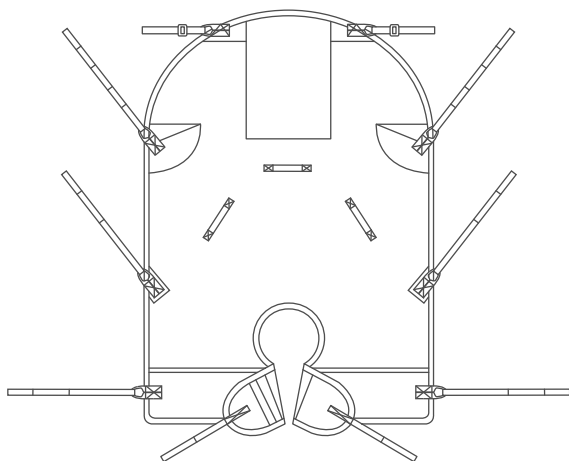

A




Shoulder circumference

B

Top of head to base of spine

Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921509-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921509-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921509-M
	Large	42.5–51.25in (108–130cm)	31.5–34.5in (80–88cm)	M921509-L
	XLarge	51.25–56.75in (130–144cm)	34.5–39.5in (88–100cm)	M921509-XL
	XXLarge	56.75–63in (144–160cm)	38–41in (96–104cm)	M921509-XXL

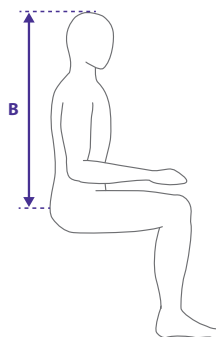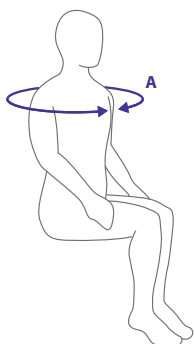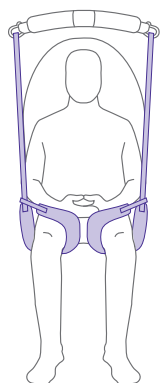


Guidelines

- SWL 272kg / 600lb
- Wash 95°C / 203°F
- Low Tumble Dry



In-Situ Deluxe Leg Sling

WITH LOOPS

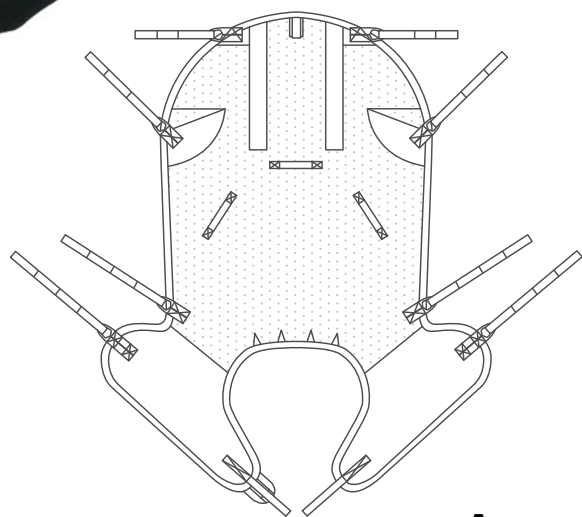

- The In-Situ Deluxe Leg sling provides full body support on transfer and offers good hip, thigh and femur support. This sling includes integral head support with removable support bones and hip tapes for greater user comfort.
- This divided leg sling is easy to apply, with low-friction fabric to leg areas. The sling includes looped hip tapes offering optimum pick-up options and enhanced user comfort by eliminating pressure from the hip area during lifting.
- In-Situ design may allow sling to be left in place for longer periods with full written risk assessment.
- The sling is perfect for standard transfers but also can be used in bathing transfers as the fabric is a quick drying spacer mesh. Some commode access available, with tailoring to the shoulders, head and commode areas giving a superior fit for even complex body shapes. This sling is suitable for asymmetrical body shapes as the fabric helps to mold to individual body shapes.
- **Superfine PLUS™** (100% polyester spacer fabric; superior comfort over standard sling fabrics; easy application, suitable for bathing.)

A




Shoulder circumference

B

Top of head to base of spine

Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921511-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921511-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921511-M
	Large	42.5–51in (108–130cm)	31.5–34.5in (80–88cm)	M921511-L
	XLarge	51–56.5in (130–144cm)	34.5–39.5in (88–100cm)	M921511-XL
	XXLarge	56.5–63in (144–160cm)	38–41in (96–104cm)	M921511-XXL

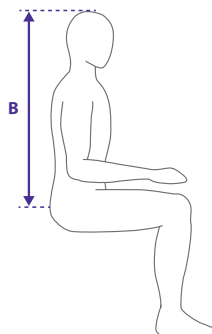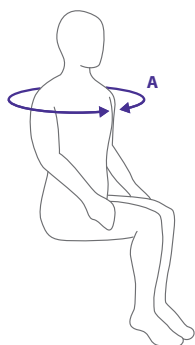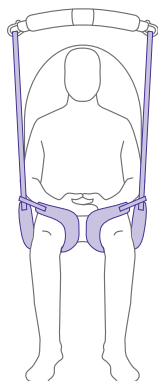

Guidelines


- SWL 272kg / 600lb
- Wash 95°C / 203°F
- Low Tumble Dry



In-Situ Recline Sling

WITH LOOPS

- The In-Situ full body Recline sling comes with a padded roll for user comfort under the leg section and a pommel strap for additional user safety during the transfer.
- The inclusion of the hip tapes provide additional support to the hip area, reducing pressure while lifting occurs.
- The In-situ Recline sling also makes a good amputee sling due to the transfer position and pommel strap.
- In-Situ design may allow sling to be left in place beneath the user for longer periods with full written risk assessment.
- **Superfine PLUS™** (100% polyester spacer fabric; superior comfort over standard sling fabrics; easy application, suitable for bathing.)

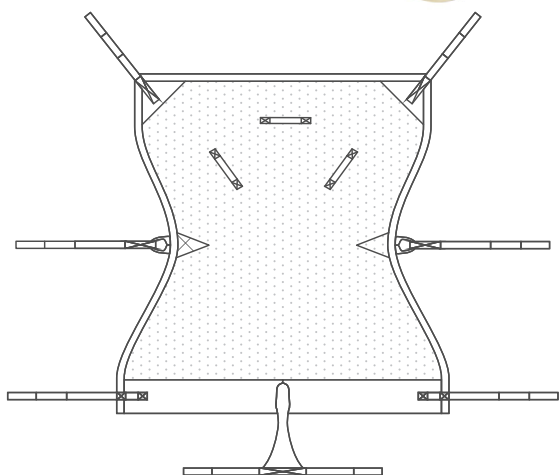

A




Shoulder circumference

B

C7 to back of knee

Product Codes

	XSmall	30–34in (76–86cm)	32–36.25in (81–92cm)	M921507-XS
	Small	34–38in (86–96cm)	36.25–41in (92–104cm)	M921507-S
	Medium	38–42.5in (96–108cm)	41–45.75in (104–116cm)	M921507-M
	Large	42.5–51in (108–130cm)	45.75–50.5in (116–128cm)	M921507-L
	XLarge	51–56.5in (130–144cm)	50.5–55.25in (128–140cm)	M921507-XL
	XXLarge	56.5–63in (144–160cm)	55.25–60in (140–152cm)	M921507-XXL

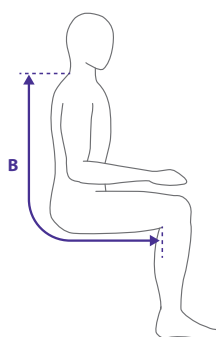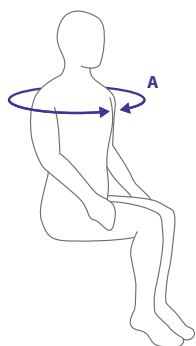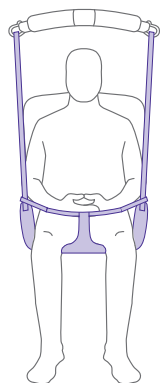

Guidelines


- SWL 272kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



Walking Sling

WITH LOOPS

- Fitted adjustable slider tapes
- Simple to apply padded harness
- Additional padding to under arm sections
- Adjustable groin straps with removable comfort covers
- Designed to assist with standing/walking
- Some weight bearing ability recommended

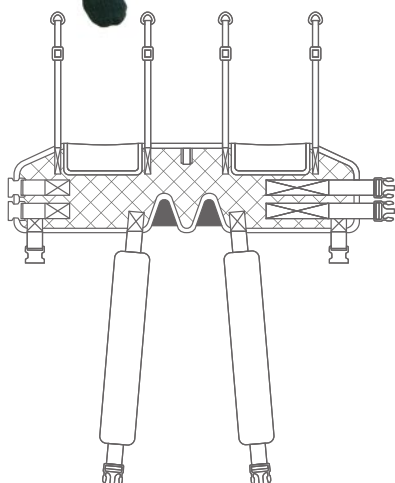

A



Waist circumference

B

Armpit to Waist

Product Codes

	Small	34.25–42in (87–107cm)	10in (25cm) or over	M921503-S
	Medium	38.25–46in (97–117cm)	10.5in (27cm) or over	M921503-M
	Large	44–52in (112–132cm)	11in (28cm) or over	M921503-L

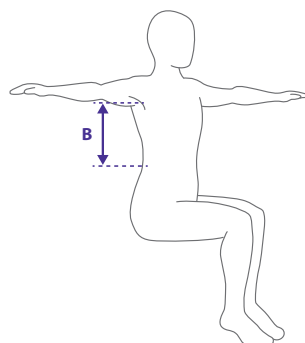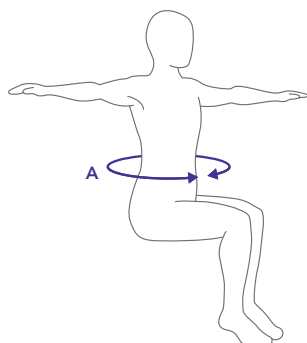

Guidelines

- SWL 272 kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



Repositioning Sling

SHEET STYLE


- The sling can be used in a side-lying position to facilitate a 'log-roll' movement which will give access for personal care, to change patient's position for help in assisting with pressure management and is also useful for changing soiled bedding. Can be used with a slide sheet positioned underneath to help with the glide and to ensure that the user is positioned correctly back into the center of the bed after the turn.
- With sling attached to the spreader bar, user can be lifted slowly and leveled correctly before repositioning. This procedure will help to assist the caregiver in repositioning the user higher up in bed.
- The positioning sling comes with eight tapes plus two head support tapes. When fully attached to the lift, this creates a supine transfer sling useful for lateral transfers from a bed/stretcher to an operating theater or diagnostic table.
- Available in the following material types:
Mesh (100% polyester open weave mesh; high density, durable, used for bathing as quick drying.)
Poly (100% polyester; easy application, glides and slides smoothly, dries quickly after washing.)

* Available as a disposable sling model for single-patient use (M921518)



Product Codes

	Mesh	Poly
One size	M921506	M921522

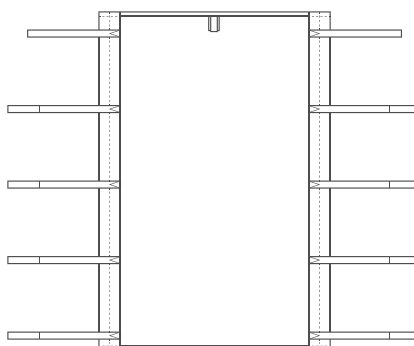

Guidelines


- SWL 454kg / 1000lb
- Wash 95°C / 203°F
- Low Tumble Dry



Stand Assist Sling

"ROLL" STYLE

- A good starter 'Roll' Stand Aid sling that sits under the arms. This stand assist sling is padded to provide additional comfort and stability. Height-adjustable for multi-patient application. Hook and loop fixing chest support with heavy-duty, side-release buckle for user safety. Ideal for maintaining mobility or rehabilitating users who have suffered injury and need to gradually build standing strength.
- User must have reasonable weight-bearing ability.
- Adjustable loop fixing. 100% Quilted Polyester.

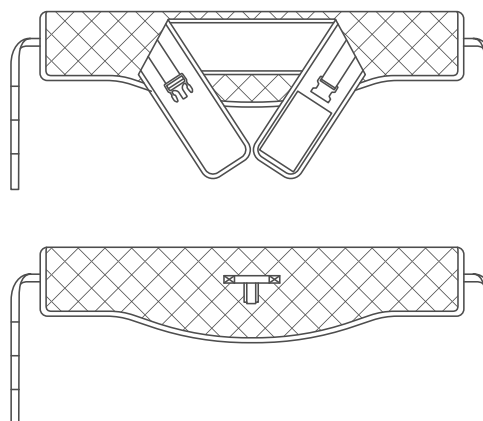

A

Waist circumference

Small / Medium

38.25–46in (97–117cm)

Large / XLarge

49.25–57in (125–145cm)


Product Codes

D202MEDLP

D202XLGLP

Guidelines

- SWL 227 kg/500lb
- Wash 95°C / 203°F
- Low Tumble Dry


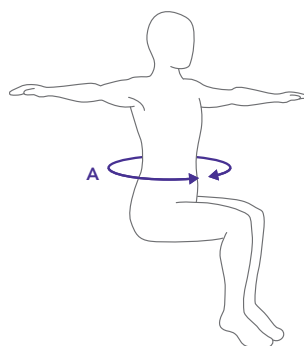




Stand Assist Standard

WITH LOOPS

- Fully padded with a boned back for added support
- Padded base and under arm rolls for extra comfort
- Hook & loop fastening with clunk-click buckles for user safety
- Ideal for maintaining mobility and for building strength
- Weight bearing ability required
- Adjustable loop fixing. 100% Quilted Polyester.

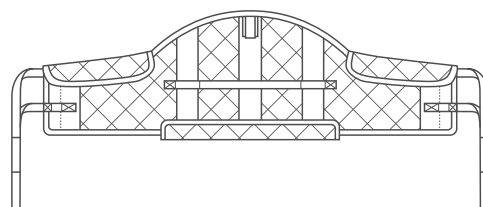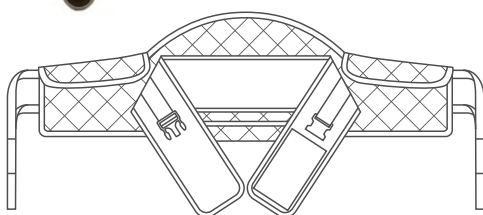


A

Waist circumference

Small	34.25–42in (87–107cm)	M921519-S
Medium	38.25–46in (97–117cm)	M921519-M
Large	44–52in (112–132cm)	M921519-L
XLarge	49.25–57in (125–145cm)	M921519-XL

Product Codes

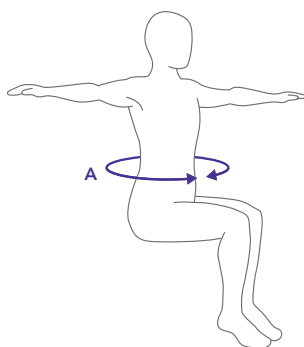

Guidelines



- SWL 272 kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry



Stand Assist Sling

CORSET STYLE

- Ideal for maintaining mobility or rehabilitating users who have suffered injury and need to gradually build standing strength.
- Very supportive corset style Stand Assist Sling with elasticated central panel in the back of the sling which provides additional support.
- Soft waist support with hook and loop fixing and two sets of heavy-duty, side-release buckles for user safety. Padded underarm rolls for extra comfort. Suitable for those with sufficient weight-bearing ability and core stability. Great general stand assist sling.
- Adjustable loop fixing. 100% Quilted Polyester.

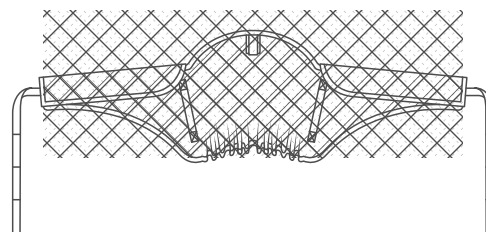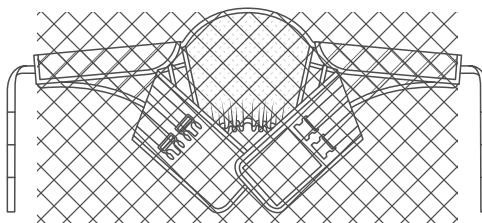


A

Waist circumference

Small	34.25–42.25in (87–107cm)	M921516-S
Medium	38.25–46in (97–117cm)	M921516-M
Large	44–52in (112–132cm)	M921516-L
XLarge	49.25–57in (125–145cm)	M921516-XL

Product Codes

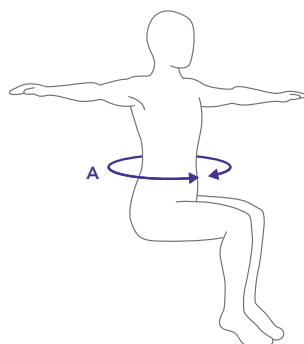

Guidelines



- SWL 227 kg/500lb
- Wash 95°C / 203°F
- Low Tumble Dry

Stand Assist with Clips

WITH CLIPS



- Semi rigid corset style stand assist
- Deep back design providing additional support
- Clunk-click buckle and hook and loop fixing waist support for additional safety
- Suitable for maintaining mobility or rehabilitation
- Padded underarm rolls for extra comfort
- User should have some core stability and weight bearing ability
- Clip fixing. 100% Quilted Polyester.

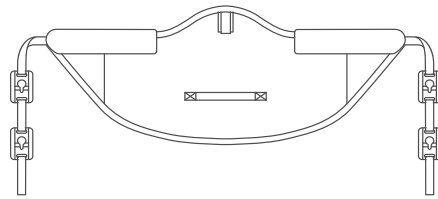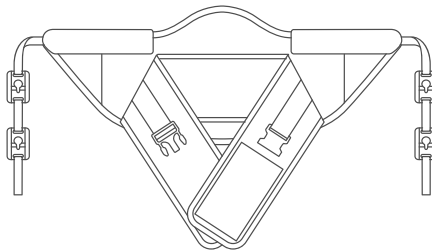


A

Waist circumference

Product Codes

Small	34.25–42in (87–107cm)	M921525-S
Medium	38.25–46in (97–117cm)	M921525-M
Large	44–52in (112–132cm)	M921525-L
XLarge	49.25–57in (125–145cm)	M921525-XL

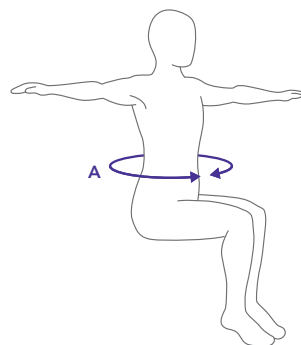


Guidelines

- SWL 272 kg/600lb
- Wash 95°C/203°F
- Low Tumble Dry

High Easy Disposable Sling

WITH LOOPS

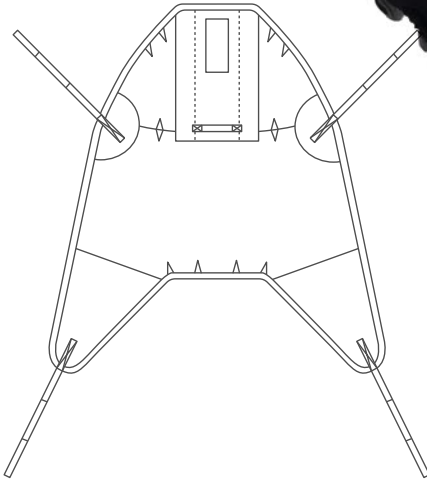

- The Disposable single patient use sling can be used as a general purpose sling for short stay hospital or residency visits, and features a sewn-in, stiffened head support.
- Easy to apply, all supplied with padded legs and additional tailoring to commode aperture as standard.
- Used specifically in acute settings to limit cross contamination.
- Designed for one patient use only and for the duration of a short stay. If the sling shows any signs or deterioration or is soiled, it should be destroyed immediately.
- Suitable for use with the leading standard spreader bar type lifts (loop fixing). Please see compatibility statement at the back of this guide.
- All slings are fitted with a warning 'DO NOT USE' label which only becomes visible when in contact with liquid as an additional safety measure.
- Sold in packs of ten.
- 95% Non-Woven Polypropylene / 5% Polyester (Not suitable for wet environments.)

A




Shoulder circumference

B

Top of head to base of spine

Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921505-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921505-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921505-M
	Large	42.5–51.25in (108–130cm)	31.5–34.5in (80–88cm)	M921505-L
	XLarge	51.25–56.75in (130–144cm)	34.5–39.5in (88–100cm)	M921505-XL
	XXLarge	56.75–63in (144–160cm)	38–41in (96–104cm)	M921505-XXL

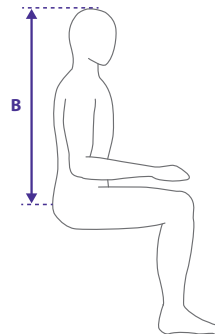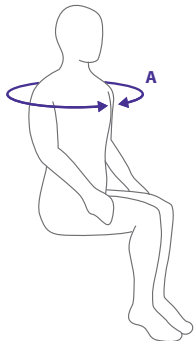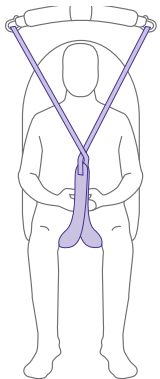


Guidelines

- SWL 272kg/600lb
- DO NOT WASH
- Short service disposable sling
- Not for wet environments



High Easy Disposable Sling

WITH CLIPS

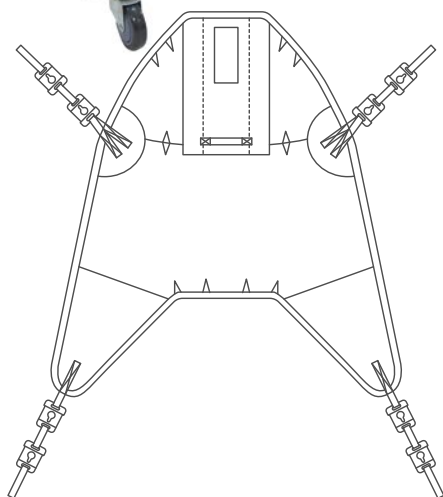

- The Disposable single patient use sling can be used as a general purpose sling for short stay hospital or residency visits, and features a sewn-in, stiffened head support.
- Easy to apply, all supplied with padded legs and additional tailoring to commode aperture as standard.
- Used specifically in acute markets to limit cross contamination.
- Designed for one patient use only and for the duration of a short stay. If the sling shows any signs or deterioration or is soiled, it should be destroyed immediately.
- Suitable for use with the leading standard stud spreader bar type lifts (clip fixing). Please see compatibility statement at the back of this guide
- All slings are fitted with a warning 'DO NOT USE' label which only becomes visible when in contact with liquid as an additional safety measure.
- Sold in packs of ten.
- 95% Non-Woven Polypropylene / 5% Polyester (Not suitable for wet environments.)

A




Shoulder circumference

B

Top of head to base of spine


Product Codes

	XSmall	30–34in (76–86cm)	23.5–26in (60–66cm)	M921512-XS
	Small	34–38in (86–96cm)	26–29in (66–74cm)	M921512-S
	Medium	38–42.5in (96–108cm)	29–31.5in (74–80cm)	M921512-M
	Large	42.5–51.25in (108–130cm)	31.5–34.5in (80–88cm)	M921512-L
	XLarge	51.25–56.75in (130–144cm)	34.5–39.5in (88–100cm)	M921512-XL
	XXLarge	56.75–63in (144–160cm)	38–41in (96–104cm)	M921512-XXL

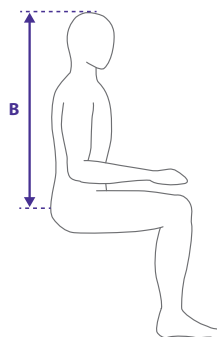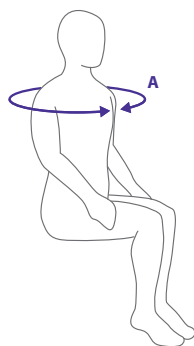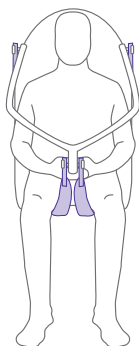

Guidelines


- SWL 272kg/600lb
- DO NOT WASH
- Short service disposable sling
- Not for wet environments

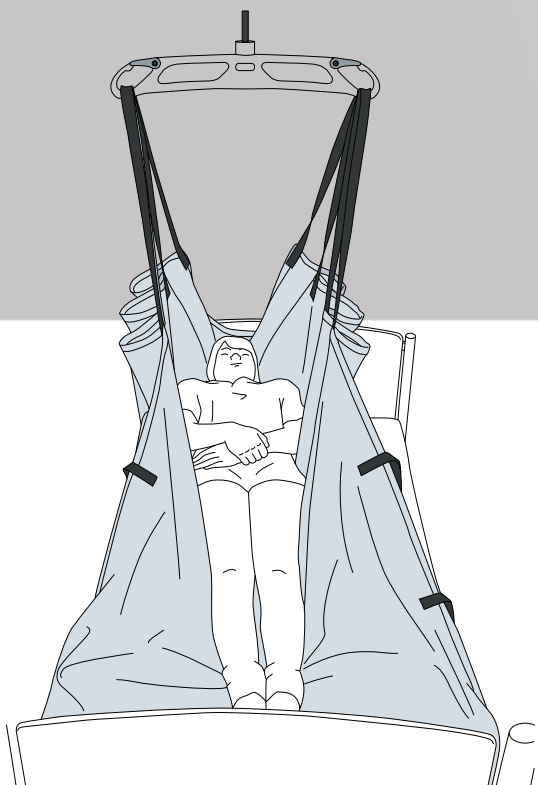

Disposable Repositioning Sling

WITH LOOPS

- This sling is designed as a NO WASH, single patient use product.
- Intended for lateral transfer of patients, moving patient up in the bed and turning.
- Note: The repositioning sling must not be used for moving and lifting in a sitting position
- Disposable use to eliminate the challenges associated with cross infection control.
- Full risk assessment required for suitability
- Polyester webbing tapes

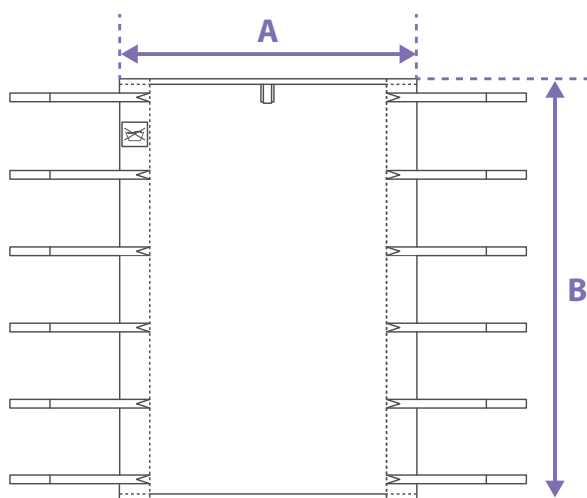

AxB
Dimensions

One size

35x75in (89x190cm)

Product Codes

M921518

Guidelines

- SWL 272kg / 600lb
- DO NOT WASH
- DO NOT DRY

Sling & Lift Compatibility

The Silvalea universal sling series has been selected and manufactured specifically to be interchangeable with a variety of lifts, as well as being the perfect partner for the Savaria line of lifters. The products are intended as replacement slings that are easy to fit while providing superior support.

All slings should be chosen following a thorough assessment by a competent healthcare professional, checking carefully that the sling chosen is the right style and size for the service user. A visual check of the sling is vital before each use. Slings that are damaged should be withdrawn and destroyed. Do not attempt to lift any service user if the sling is compromised.

Follow the sling label for washing guidelines and safe working load advice. If in doubt consult your healthcare professional before use.

**Please contact your local Span representative
to order sling products, or for more information**



spanamerica.com

Silvalea slings are distributed in North America by Span (USA: Span-America Medical Systems, Inc, Canada: Span-Canada Medical Products LP).
Span America: 70 Commerce Center, Greenville, SC 29615



silvalea.com

Silverhills Buildings, Decoy Ind. Estate, Newton Abbot, Devon, TQ12 5LZ, UK



Silvalea Ltd, Span-America Medical Systems, and Span-Canada Medical Products LP are divisions of Savaria Corporation.

For more information about Savaria patient lifts: **patientcarelifts.com**

savaria.com