

Instructions of Use

Stand Assist Roll Style Sling

STANDARD RANGE

Reference: **SA(R) - IOU**

Product Code/s: **D202**

Revision: **04.06.2021**


Copyright & Design Rights: **Silvalea.**



WARNING



Do NOT use clip sling attachments on a loop type spreader bar and Do NOT use loop sling attachments on a stud type spreader bar.

LOOPS

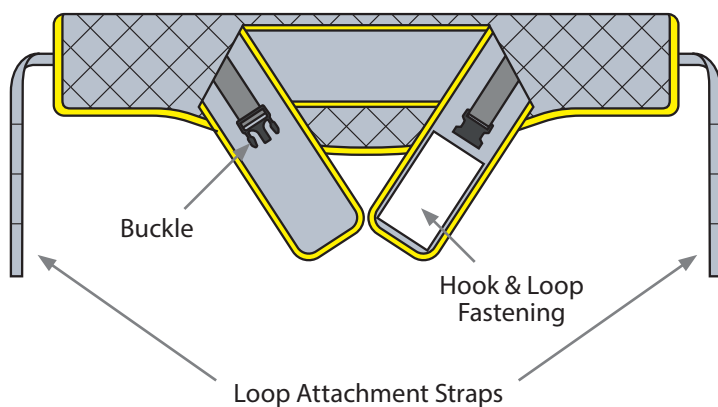


Loop Sling Attachments - Loop Type Spreader Bars:


Please note there are many variations of the loop type spreader bar but they should all be used exclusively by slings with: **LOOP SLING ATTACHMENTS**

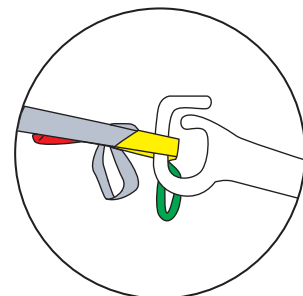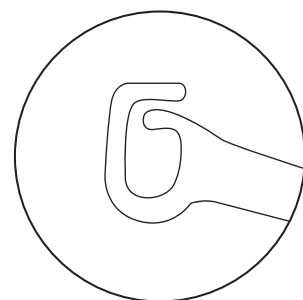


Sling Application:

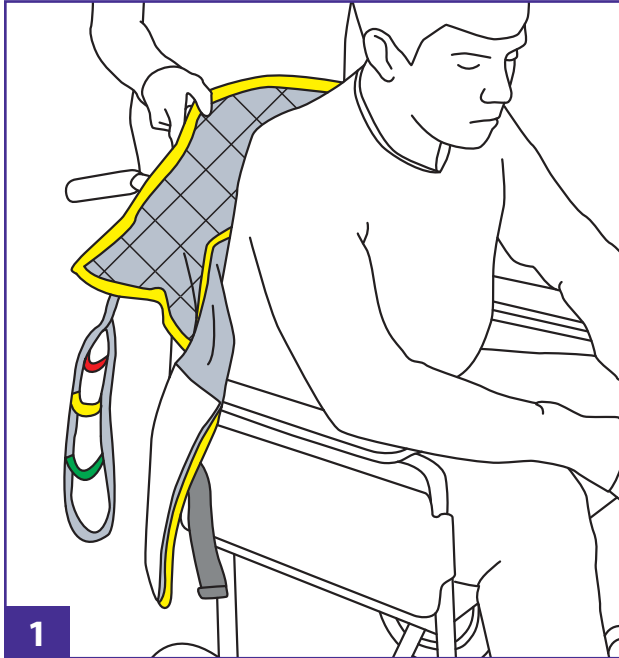


Open chest support and place the sling around the back of the patient.

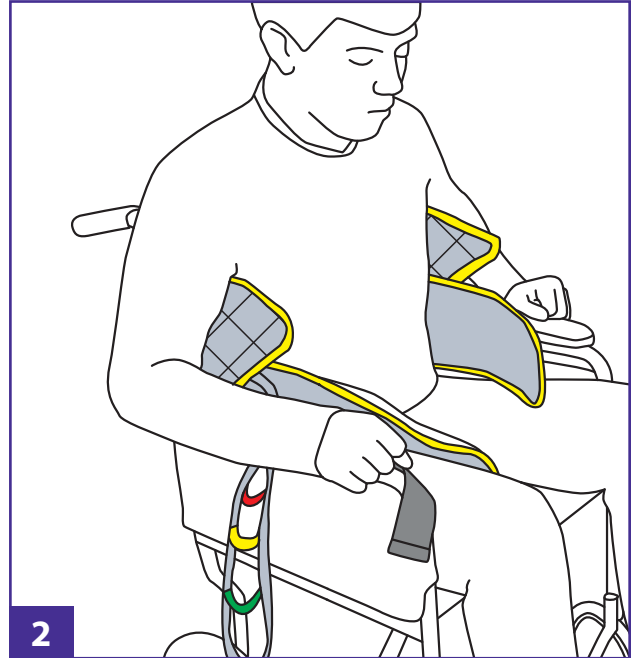

Take the patient's arms and place them over the sling.

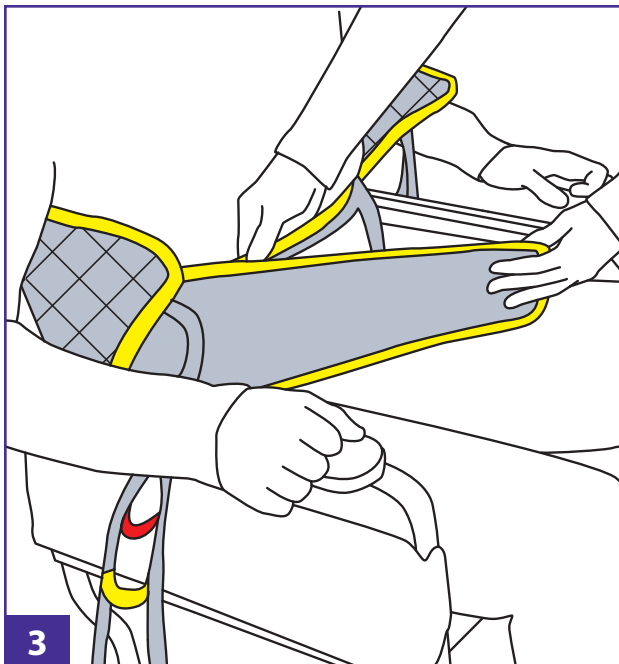


Fasten the hook & loop chest support around the patient's chest (for a women this should be just under the bust).

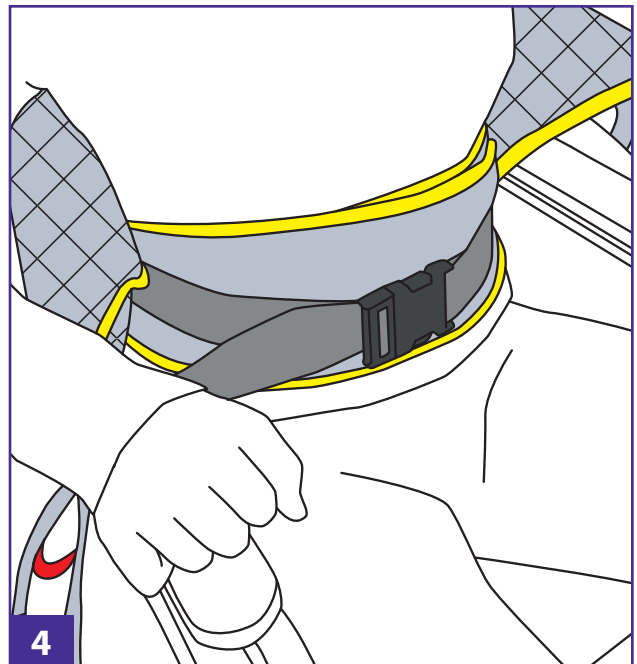


Fasten the waist support buckles and tighten appropriately.

Sling Application Continued:

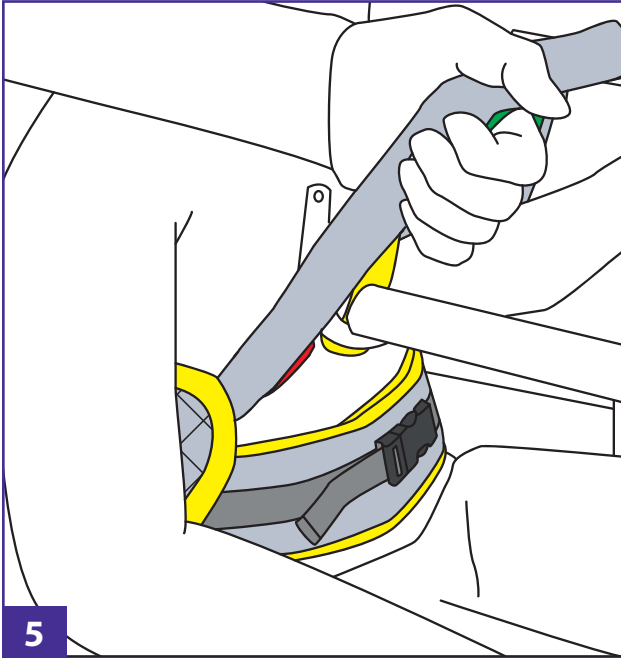

Attach the corresponding loop straps to the fixings on each side of the stand aid.

When securing the loops always double check that the loops are secure on the fixings before commencing transfer.

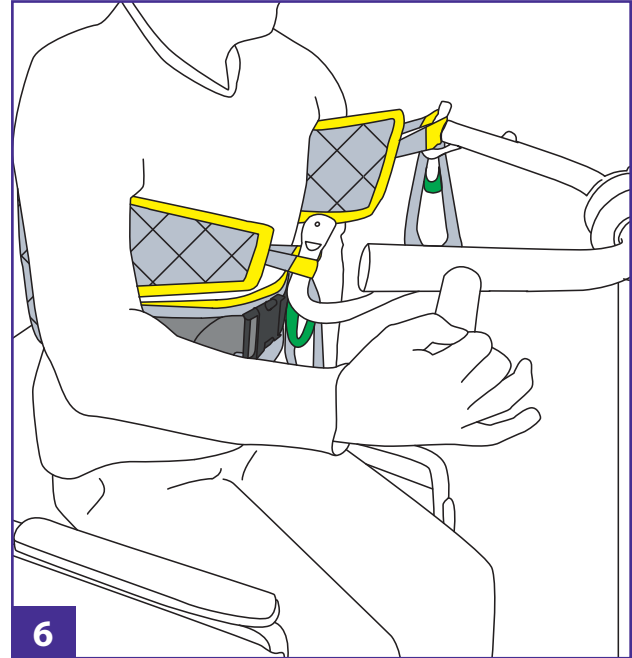

Once you are happy that the sling is securely attached to the lift and is the correct size and fit for your patient the transfer can begin.

Once the patient has been raised slightly check and tighten buckles on waist support, if necessary.



General Instructions:

The Stand Assist Roll Style Sling is designed for a general purpose transfer for weight bearing patients from a seated position to standing.

The patient must be assessed by a competent person who is fully trained in the suitability, application and fitting of the sling.

Carers should always be trained in the use of lifts and slings. Correct sizing of a sling is important, if in any doubt consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products should be checked each time they are used for any signs of deterioration.

Please Check:

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the Lift
- safe working load of both the lift and sling /product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.

The Service Label is a visual indicator of when the 6-monthly or annual lift inspection has been completed.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- DO NOT IRON

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convactor heater or steam pipes.



SWL 272kg/600lb/42st

For sizing guides and images of this product please visit **www.silvaleaslings.com**
and for further assistance, contact your Silvalea distributor



Silvalea Ltd silvalea.com
Silverhills Buildings. Decoy Ind. Estate, Newton Abbot, Devon, TQ12 5LZ. UK



Distributed by Span-America
Span-America; Span Medical Products Canada: spanamerica.com



Distributed by Savaria: savaria.com / patientcarelifts.com



ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013. Malta

