

SPAN Geo-Matt® Cushion

(4" Medium Density)



- Clinically proven Geo-Matt design featuring individual cells that respond to weight as necessary
- Ring-of-Air® improves air flow to reduce heat and moisture

IMPORTANT INFORMATION – READ BEFORE USE

Place cushion in chair with zipper side to the back and vinyl fabric side facing down into chair upholstery as indicated by "Bottom rear" notation on cushion bottom. The SoftSkin cover is quite durable. However, if it does develop a tear, patch it with waterproof tape. If you should need to replace the entire SoftSkin, contact your original Geo-Matt source or call Span-America's Customer Service Department at 800-888-6752 for assistance.

The SoftSkin can be wiped clean with warm water and soap if it becomes soiled. If the Geo-Matt cushion itself needs washing, DO NOT immerse it completely. SPOT CLEAN ONLY, using warm water and a mild detergent.

You may cover the Geo-Matt with a light covering of your choice.

If underpads, Chux, etc. are used, do not layer them. Do continue to use these items as necessary but be aware that their use may interfere with the Geo-Matt wheelchair cushion's optimum performance.

Model Number	Size (WxL)	Description
50746-040	18" x 16"	Cushion only
50746-050	18" x 16"	With Soft Skin® sleeve
50746-051	18" x 16"	With fire-retardant Soft Skin® sleeve
50746-084	18" x 16"	With anti-slip cover

Span-America Medical Systems 70 Commerce Center Greenville, SC 29615 USA



Advena Ltd.
Tower Business Centre, 2nd Flr
Tower Street, Swatar
BKR 4013 Malta



Advena Ltd.
Pure Offices, Plato Close
Tachbrook Park, Warwick,
CV34 6WE United Kingdom



Australian Sponsor:
KD&A Pty Ltd
(ACN 114 941 338)
286 Flinders Street
Adelaide, South Australia, 5000

